

i The shortcuts included in this job aid are applicable to users in the United States and Canada at organizations where the superset resources are installed.

Default Mouse Button Behavior for Imaging Viewing


Button	Action
Left button press + Hold	Scroll/Page
Middle button press + Hold	Zoom
Middle button scroll	Scroll
Left + Right button press + Hold	Pan
Right button press + Hold	Window Level
Right button single click	Open context/series menu

Mouse Customization for Imaging Viewing

Users can personalize the following mouse buttons via

Navigator Options > User Preferences:

- Left button
- Middle button
- Left and right combination

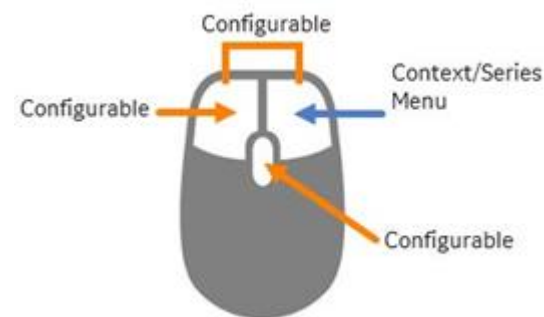

i Site administrators can configure a default set of tools for the mouse buttons for all users at a site.

Universal Viewer 8.0 Default Keyboard Shortcuts for Clinicians

Navigation

Action	Shortcut
Navigator	N
Patient Folder	F1

Zoom

Action	Shortcut
Zoom Mode	Z
Set Zoom to 1	1
Set Zoom to 1.5	5
Set Zoom to 2	2
Set Zoom to 2.5	Ctrl + 2
Set Zoom to 3	3
Set Zoom to 3.5	Ctrl + 3
Zoom to 4	4
Undo the Zoom	6
Zoom to ROI	Ctrl + Z

Hanging Protocols

Action	Shortcut
Next HP	H
Previous HP	J

Image Manipulation

Action	Shortcut
Activate Arrow	A
Cine	C
Flip	F
Global Stack	G
Key Images (Quick Add) (mark/unmark key images)	K
Invert	V
Lens	L
Measurements	M
Mirror	9
Quick Save Presentation State	Q
Reset	R
Rotate 90° (clockwise)	0 (Zero)
Sharpen	S
Temporary Measurement	T
Toggle Overlay (show/hide)	O
Triangulation	F2

Paging

Action	Shortcut
Activate Page/Scroll Mode	P
Activate Scroll Options	Ctrl + Alt + S
Cine Toolbar - Next Page	Page Down (Next)
Cine Toolbar - Previous Page	Page Up (Prior)
Go to First Image/Page	Home
Go to Last Image/Page	End
Go to Middle Image/Page	Insert
Next Page	Right arrow
Previous Page	Left arrow
Next Series	Down arrow
Previous Series	Up arrow
Next Preset Window	Spacebar

Save/Quit

Action	Shortcut
Done	F12 or D
Exit	X